

# OPERATING MANUAL



**RG72**

## SAFETY NOTICE & CAUTION


- a) Consult your doctor, therapist or other qualified professional to determine if this Rollator RG72 will suit your particular needs.
- b) For proper positioning and use of your Rollator, imagine a line drawn across the back of the Rear Wheels. Part of one foot should always be slightly ahead of that line as you walk
- c) This rollator is for individual use only.
- d) Rollator is designed for indoor & outdoor use.
- e) Regularly check the brakes for proper operation. Pulling up on the brake handle will slow the rollator. Pushing down on the brake handle will lock the wheels. Pull up on the brake handle to release the brake lock. **(refer "BRAKE USAGE" on page 2)**
- f) Do NOT use as a wheelchair or transport chair.
- g) Rollator must be fully opened and locked position when being used as a walking aid.
- h) Wear a proper shoes while using this rollator
- i) Follow the instruction "**OPENING/FOLDING**" to fold or unfold the rollator **(refer page 2)**.
- j) Do NOT sit on rollator and push yourself around using your feet to propel.
- k) Do NOT stand on Seat .
- l) Before sitting on seat, be sure the rollator is fully opened, and the brake is in the locked position.
- m) Do NOT use on stairs or escalators.
- n) Do NOT seat on the rollator if the ground is slopped or uneven.
- o) Exercise caution when walking on sloped surfaces.
- p) Do not hang objects from the backstrap.
- q) Do not use if the rollator is not functioning properly.
- r) Maximum user weight 250lbs / 113kg .
- s) When storing/transporting your rollator do not anything on the rollator, as damage may occur.

## ASSEMBLY

### ATTENTION!

Ensure that all bolts are securely fastened before using this rollator!  
Brakes must be in the parked /locked position before sitting on the rollator!

### 1. Install the brake handles

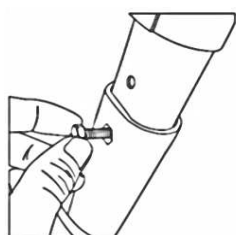


**Rollator should be in the unfolded position before beginning.** Slide the Brake handle into the frame and insert height adjustment screw to desired height. Align hexagon shape of bolt into the hexagon shape in the frame.




From the outside of the frame tighten the black adjustment knob onto the screw

The consequences such an adjustment may have on the stability.

### 2. Install the backstrap

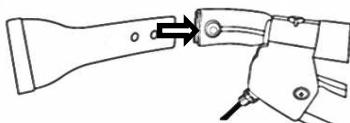


Slide backstrap into the backrest connector. The padded side of the backstrap should face the inside of the rollator.

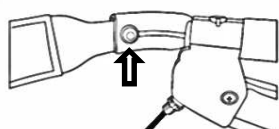

Secure the backstrap into position by tightening the screw and nut.

### 3. Install the basket

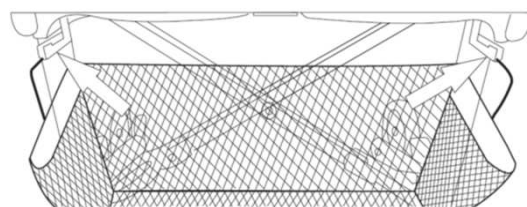

Slide the basket hooks into the brackets located under the seat at the front.


**CAUTION: DO NOT OVERLOAD BASKET MAX load 11lbs/5kg**

### PRODUCT LABEL


1. Product name
2. CE-marked
3. Read the manual
4. Max width
5. Max user weight/basket load
6. Indoor and outdoor use
7. Do not use as wheelchair

1	<b>RG72</b>
2	<b>CE</b>
3	
4	
5	Max user weight: 113kg/250lbs Basket load: 5kg/11lbs
6	
7	


## BRAKE USAGE

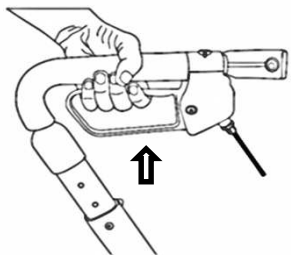

Pull up on brake handle to slow down or to release the parking brake.

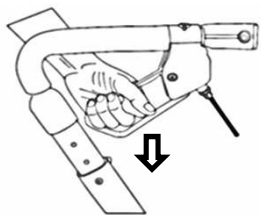

Push down on brake handle to lock the brake into PARK.

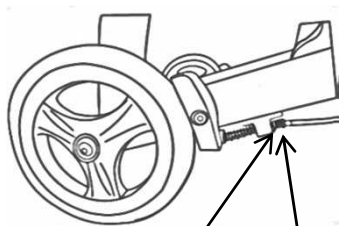


Adjust brake tension, Loosen the locking nut, and turn the adjustment screw clockwise/ counter clockwise

locking nut

adjustment screw

## OPENING/ FOLDING

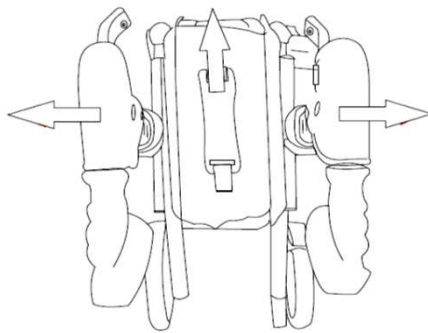

↑ To fold a rollator, pull up on the red handle

← → To unfold rollator. Pull handles apart, and push down on the seat to lock into place.

## USAGE



Do not support your weight on the handle attempting to sit. Use the handles to stabilize your body as you carefully turn around. Keeps your hand on the handles and slowly lower your weight onto the seat

## STORAGE AND TRANSPORTATION

Remove the basket, and fold your rollator for easier storage and transportation. Do not rest anything on your folded rollator when being stored or transported as damage may occur.

## MAINTENANCE

- Clean the rollator using mild soap and water. Wipe dry.
- Keep wheels clean.
- Check screws and handle height adjustments are tight.
- If your rollator is not working properly. Contact your dealer.
- DO NOT USE if the rollator is not functioning properly.
- Use only authorized parts/accessories with your rollator.



## WARRANTY

TROLLI MASTER warrants the Journey (RG72) against defects in material and/or workmanship for a period of one year from date of purchase. Those items no warranted are wear items, such as handgrips, brake cables, and wheels. TROLLI MASTER will not warranty the rollator if it has been damaged due to improper installation, modification, and/or misuse. Service/Labour charges are the discretion of the dealer, and are not the responsibility of Trolli Master.


TROLLI MASTER's sole liability extends to repair or replacement of parts or components as previously specified. The foregoing is in lieu of all warranties expressed, implied or statutory whether written or oral. TROLLI MASTER shall not be liable for any sequential or incidental damages whatsoever.

### Manufactured By :-

TROLLI MASTER SDN. BHD. (209382-P)  
Lot 525, Mukim Telok Panglima Garang, 42500 Daerah Kuala Langat,  
Selangor Darul Ehsan, Malaysia.

Tel : +60(3) 3191 4077 , 3191 2837  
Fax : +60(3) 3191 2152 , 3191 3022

Email : tm@trollimaster.com.my  
Website : www.trollimaster.com

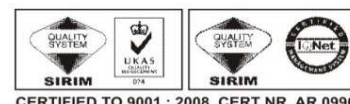

CERTIFIED TO 9001 : 2008 CERT NR. AR 0996